

Clinell Alcohol Wipes Efficacy Summary

| Effective against bacteria | Test | Log Reduction | Conditions | Contact Time |
|-----------------------------------------------------------|----------|---------------|------------|--------------|
| <i>Acinetobacter baumannii</i> | EN 13727 | >5 | Dirty | 60 seconds |
| <i>Enterococcus hirae</i> | EN 13727 | >5 | Dirty | 30 seconds |
| | EN 16615 | >5 | Dirty | 30 seconds |
| <i>Escherichia coli k12</i> | EN 13727 | >5 | Dirty | 120 seconds |
| <i>Pseudomonas aeruginosa</i> | EN 13727 | >5 | Dirty | 30 seconds |
| | EN 16615 | >5 | Dirty | 30 seconds |
| <i>Salmonella typhimurium</i> | EN 13727 | >5 | Dirty | 60 seconds |
| <i>Staphylococcus aureus</i> | EN 13727 | >5 | Dirty | 30 seconds |
| | EN 16615 | >5 | Dirty | 30 seconds |
| Methicillin resistant <i>Staphylococcus aureus</i> (MRSA) | EN 13727 | >5 | Dirty | 60 seconds |
| Effective against Mycobacteria | Test | Log Reduction | Conditions | Contact Time |
| <i>Mycobacterium avium</i> | EN 14348 | >4 | Dirty | 2 minutes |
| <i>Mycobacterium terrae</i> | EN 14348 | >4 | Dirty | 2 minutes |
| Effective against viruses | Test | Log Reduction | Conditions | Contact Time |
| Adenovirus | EN 14476 | >4 | Dirty | 30 seconds |
| Norovirus | EN 14476 | >4 | Dirty | 30 seconds |
| Murine Coronavirus | EN 14476 | >4 | Dirty | 30 seconds |
| Vaccinia virus | EN 14476 | >4 | Dirty | 15 seconds |
| Effective against yeast | Test | Log Reduction | Conditions | Contact Time |
| <i>Candida albicans</i> | EN 13624 | >4 | Dirty | 30 seconds |
| | EN 16615 | >4 | Dirty | 30 seconds |